# Lansinoh.

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**Storing and thawing your breastmilk**

Prior to storing and handling breastmilk, wash hands well with soap and water before use. Breastmilk should be stored in a clean, sealed container.

**How long to store breastmilk**

Depending on the country, breastmilk storage guidelines may vary. Please refer to your local country expressed milk storage guidelines for healthy term babies (check with your healthcare professional, if needed).

**Caution**

- Keep all components not in use out of the reach of children.
- Use of other sanitising methods (e.g., microwave) is not recommended and may damage some components.
- Do not clean, store or allow to come into contact with solvents or harsh chemicals.
- Damage could result.
- When not in use, store in a dry and covered place.

DO NOT store your breastmilk in the fridge/freezer door. The door tends to be the warmest part of the fridge/freezer. Date all milk and use the oldest milk first.

Ideally, thaw frozen milk in the refrigerator overnight, or place the container under cool running water, gradually warming the water (no more than 37°C/98°F), until the milk is thawed. Once the milk is liquid, but still chilled, dry the bottle and refrigerate (at least 4°C) until use.

Prior to use, gently swirl the bottle of thawed breastmilk before feeding to baby to mix the layers that have separated. It is normal for expressed milk to look thin, bluish, yellowish or even brownish in colour and for it to separate into a milk layer and a cream layer when stored. Some mothers notice their milk smells slightly soapy once defrosted, due to high levels of lipase in the enzyme that breaks down fat. The milk is still safe to use but some babies refuse to drink it. Heating milk to a scald (bubbles around the edges, not boiling) then quickly cooling it before freezing can deactivate the lipase. If milk smells sour do not use.

DO NOT refreeze breastmilk once it is thawed.

DO NOT MICROWAVE HUMAN MILK!

Microwaving alters the composition of the milk. Microwaving may also create "hot spots" in the milk that could burn your baby's mouth.

DO NOT thaw breastmilk in very hot or boiling water (no more than 37°C/98°F).

Once breastfeeding has been established, if you choose to use a feeding bottle to feed your expressed breastmilk to your baby, then our Breastmilk Storage Bottle is compatible with the Lansinoh® NaturalWave™ teat which enables you to maintain established breastfeeding patterns, ensuring your baby will return to your breast. Should you choose to use the Lansinoh® Breastmilk Storage Bottles with a teat, please take the following precautions:

**For your child's safety and health**

**WARNING!**

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep components not in use out of the reach of children.
- Throw away at the first signs of damage or weakness.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.

**WARNING:**

- Tooth decay in young children can occur even when nonsweatened fluids are used
- This can occur if the baby is allowed to use the bottle for long periods through the day and particularly through the night, when saliva flow is reduced or if it is used as a soother.

- Inspect all components before each use.
- Throw away at the first sign of damage or weakness. Replace the teat every 7 weeks, for safety and hygiene reasons.
- Pull the feeding teat in all directions to check the product.
- Do not leave the baby alone with drinking equipment due to the risk of choking, the baby falling over or if the product has disassembled.

- WARNING:**
- Packaging materials are not part of the product.
- For your child's safety, please remove and discard all packaging materials prior to use but read and retain the instructions for future reference.

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**Storing and thawing your breastmilk**

Prior to storing and handling breastmilk, wash hands well with soap and water before use. Breastmilk should be stored in a clean, sealed container.

**How long to store breastmilk**

Depending on the country, breastmilk storage guidelines may vary. Please refer to your local country expressed milk storage guidelines for healthy term babies (check with your healthcare professional, if needed).

**Caution**

- Keep all components not in use out of the reach of children.
- Use of other sanitising methods (e.g., microwave) is not recommended and may damage some components.
- Do not clean, store or allow to come into contact with solvents or harsh chemicals.
- Damage could result.
- When not in use, store in a dry and covered place.

DO NOT store your breastmilk in the fridge/freezer door. The door tends to be the warmest part of the fridge/freezer. Date all milk and use the oldest milk first.

Ideally, thaw frozen milk in the refrigerator overnight, or place the container under cool running water, gradually warming the water (no more than 37°C/98°F), until the milk is thawed. Once the milk is liquid, but still chilled, dry the bottle and refrigerate (at least 4°C) until use.

Prior to use, gently swirl the bottle of thawed breastmilk before feeding to baby to mix the layers that have separated. It is normal for expressed milk to look thin, bluish, yellowish or even brownish in colour and for it to separate into a milk layer and a cream layer when stored. Some mothers notice their milk smells slightly soapy once defrosted, due to high levels of lipase in the enzyme that breaks down fat. The milk is still safe to use but some babies refuse to drink it. Heating milk to a scald (bubbles around the edges, not boiling) then quickly cooling it before freezing can deactivate the lipase. If milk smells sour do not use.

DO NOT refreeze breastmilk once it is thawed.

DO NOT MICROWAVE HUMAN MILK!
Microwaving alters the composition of the milk. Microwaving may also create "hot spots" in the milk that could burn your baby's mouth.

DO NOT thaw breastmilk in very hot or boiling water (no more than 37°C/98°F).

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